

HOLD CROWNS, VENEERS, & SMALL CASTINGS
IN PLACE WITH **wondergel!**

RESTORE WITH EASE

Easy to use!

- ◆ Like having an extra helping hand
- ◆ Easier to handle small & loose fitting restorations
- ◆ Eliminates risk of contamination
- ◆ Water-soluble



\$12.95

3cc syringe
Approx. 225 uses
Item #140



Wondergel Instructions

1. Place a small amount of Wondergel inside the restoration.
2. Place restoration on die. (If excess gel is squeezed out at the marginal area, hold restoration with the finger and use air pressure to blow excess down and away from the margin.)
3. Restoration is ready to build-up or contour. (It is not recommended to remove gel prior to firing.)
4. Easily remove restoration from die.