

HOLD THOSE CROWNS IN PLACE WITH
wonderpeg INSTANT FIRING SUPPORT!

BAD CROWN DAY?

\$6.50 each

min. order of 5
10cc syringe
Item #2010

**For use with
Emax and Lava crowns!**

- 👑 No pre-drying time!
- 👑 Zero firing distortion
- 👑 Does not contaminate
- 👑 For temperatures from 400°C – 1300°C



Example with a crown

1. Using the Wonderpeg syringe, simply fill the crown, jacket, or veneer; no drying time is necessary.
2. The crown, jacket, or veneer is easily balanced on the ceramic peg and does not tilt during the firing. Wonderpeg offers zero firing distortion for temperatures from 400° C to 1300° C.
3. Simply remove with a slight twist from the ceramic peg after firing and tap off any residual Wonderpeg material.

Example with a veneer

