

CLEAN WITH EASE WHILE SAVING TIME
 THANKS TO **dr.detox**
 IONIC SPONGE OVEN CLEANER!

CLEANING MADE SIMPLY AMAZING

COMPLETE KIT

\$49.95

Formula 1 & 2
 Net Wt. 1/2 lb, 236 g (each)
 Item #200

\$27.95
 Formula 1
 Net Wt. 1/2 lb, 236 g
 Item #201

\$11.95
 Formula 2
 Net Wt. 1/2 lb, 236 g
 Item #202



No vacuum needed!

- Eliminates greening, discoloration, & bubbling in porcelain restorations.
- Extends the muffle & the thermocouple life because there is no longer a need to take the muffle to high temperatures when cleaning the oven as with old-fashioned carbon products.
- Maintains the thermocouple accuracy by removing the ions of various metals that stick to the thermocouple & distort temperatures.

Formula Removes

Formula 1

Silver Ag
 Palladium Pd
 Copper Cu
 Cobalt Co

Formula 2

Aluminum Al
 Barium Ba
 Beryllium Be
 Calcium Ca

Carbon C
 Cadmium Cd
 Cesium Cs
 Chromium Cr

Iron Fe
 Lithium Li
 Magnesium Mg
 Mercury Hg

Potassium K
 Rubidium Rb
 Sodium Na
 Strontium Sr
 Zinc Zn

Dr. Detox Instructions

- Use no vacuum!
- Use only one Dr. Detox Formula (material) at a time.
- Fill provided crucible near the top with Dr. Detox Formula 1 or Dr. Detox Formula 2 material.
- Place the crucible on the oven table.
- Be sure the oven temperature is below 400° C / 800° F.
- Pre-heat the oven for 3 minutes by increasing the temperature at the rate of 40° C / 65° F per minute to a high temperature of 700° C / 1300° F.
- Hold the oven temperature at 700° C / 1300° F for 5 minutes.
- Cool the oven for 3 minutes.
- Discard Dr. Detox material after each use.