

Established on October 10, 2025 (A/2)

I. Name: Product Name: Adhesive Tip Applicator

228

II.



Huanghua Promisee Dental Co., Ltd.
Mould Industry Garden, Empolder Section, Huanghua, 061100
Hebei, P.R. China
TEL: +86-317-8882702 FAX: +86-317-8881708



MedNet EC-REP GmbH
Borkstrasse 10 – 48163 Muenster - Germany

III. Structure, Performance and Applicable Scope of Product

Adhesive tip applicators are composed of a rod and a head, and is made of polymer materials (ABS plastic, bonding material). It's designed to handle delicate items. It features a bendable neck for hard-to-reach areas. No adhesive residue is left on surfaces. It's ideal for veneers, inlays/onlays, crowns and brackets, etc.

IV. Intended Use

Adhesive tip applicators are used to stick small parts during dental treatments and procedures.

V. Use Method

1. Take one piece of the adhesive tip applicators.
2. Adhere the restoration with the head of the adhesive tip applicator and put it into the target position.

VI.



1. Please confirm the product is clean before use.
2. Please ensure the product does not stick between the rod heads or adhere to other objects during removal.
3. It is forbidden to use when the product is damaged or deformed, and it is forbidden to use when the package is damaged.
4. This product is disposable and should be disposed of after use.

VII. Storage and Maintain Method

1. This product should be stored in a clean, dry, ventilated, room temperature, relative humidity not greater than 80%, and no corrosive gas.
2. Avoid direct sunlight.
3. Prevent external extrusion and deformation during transportation and storage.

VIII. Contraindication:

No contraindication is identified by now.

IX. Symbols

	Manufacturer		CE Mark
	Date of Manufacture		Medical Device
	Batch Code		European Union Representative
	Use-by date		Consult Instructions for use
	Caution		Model Number
	Keep Dry		Made in China
	Keep away from sunlight		Keep Upwards
	Non-sterile		Do not re-use
	Importer		ISO 13485 Certification
	Unique device identifier		